





Best[®] Double Balloon Breast Brachytherapy Applicator One World — One Source — The Best!

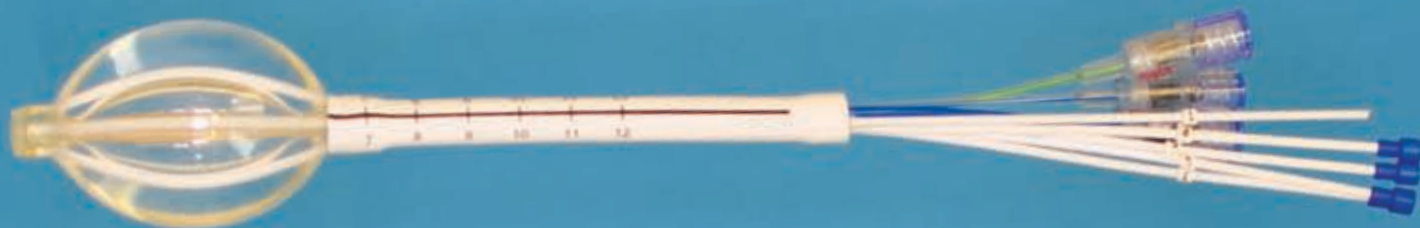

Surgical Kit includes:

- Balloon Applicator
- Treatment Catheter Covers
- Balloon Adjustment Tool
- 8 mm Trocar
- Luer Valve Aspirator
- Contrast Media Tray
- Luer Valve Covers
- (3) 60 cc Syringes
- #10 Scalpel

All items are provided in sterile pouches.

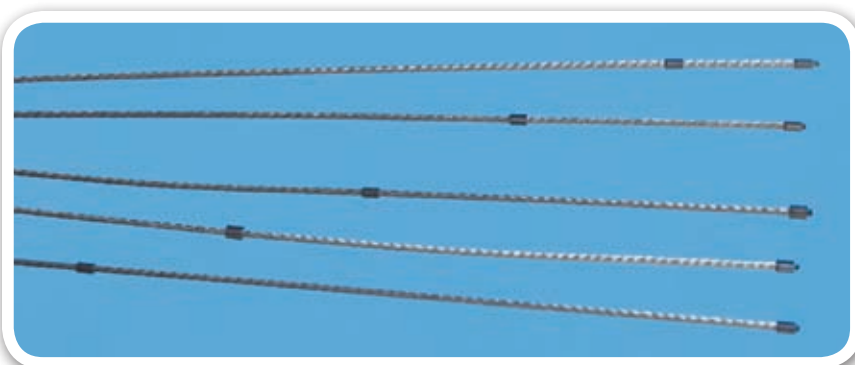

Best[®] Metallic Markers

Each set contains 5 markers for treatment catheter identification — with a metallic button on the end of each marker and a second metallic button at 1 cm, 2 cm, 3 cm, 4 cm and 5 cm.

Best Medical International, Inc. © 2016

Best Medical International, Inc. 7643 Fullerton Road, Springfield, VA 22153 USA
tel: 703 451 2378 800 336 4970 www.besttotalsolutions.com www.teambest.com

AFRICA | ASIA | EUROPE | LATIN AMERICA | MIDDLE EAST | NORTH AMERICA

The Best® Double Balloon Breast Brachytherapy Applicator offers a unique design with adjustable treatment catheters, allowing for both optimized dose delivery and conformal avoidance of critical structures.

The Best® Applicator is a single entry, multi-channel HDR accessory device. The treatment catheters can be expanded to an optimal location greater than or equal to 50% of the outer balloon diameter (> 1 cm diameter) — thereby optimizing dose delivery on a patient-by-patient basis.

Following treatment, the inner and outer balloon are collapsed, and the balloon applicator is easily removed through a 1 cm incision.

- **Adjustable treatment catheters allow for optimized dose delivery**
- **Soft, flexible design provide for patient comfort**
- **Three sizes available to provide for selection flexibility**
- **Excellent visibility on CT, MRI and Ultrasound**
- **Separate treatment catheter markers available for imaging and treatment planning**



Figure A: Model #1922 with Inner Balloon inflated to 5 ml (20 mm diameter)

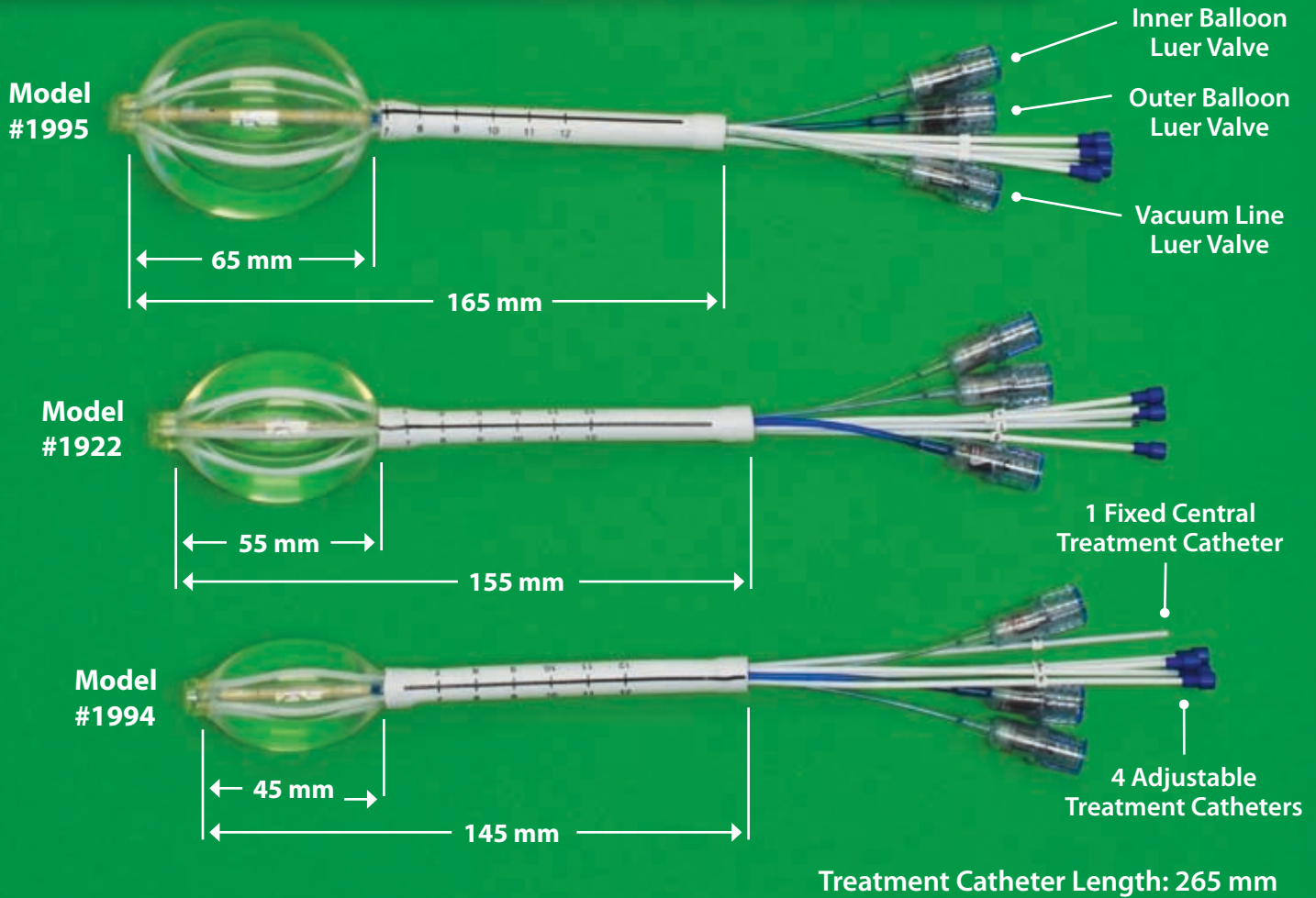


Figure B: Model #1922 with Inner Balloon inflated to 10 ml (25 mm diameter)



Figure C: Model #1922 with Inner Balloon inflated to 15 ml (30 mm diameter)

Best® Double Balloon Breast Brachytherapy Applicators — 3 sizes



Model 1994		Model 1922		Model 1995	
Outer Balloon Volume (ml)	Diameter (mm)	Outer Balloon Volume (ml)	Diameter (mm)	Outer Balloon Volume (ml)	Diameter (mm)
10	25	40	39	50	43
15	28	50	42	60	45
20	30	60	44	70	50
25	33	70	46	80	53
30	35	80	48	90	55
35	36	90	50	100	58
40	38	100	52	110	60
				120	63
				130	65
Inner Balloon Volume (ml)	Diameter (mm)	Inner Balloon Volume (ml)	Diameter (mm)	Inner Balloon Volume (ml)	Diameter (mm)
3	15	3	15	15	25
5	20	5	20	20	32
7	23	10	25	25	38
		15	30	30	42



Best[®] medical international

**Localization Needles, Fiducial Markers...and much more!
One World — One Source — The Best!**

Best[®] Breast Localization Needles



Best provides localization needles with Iodine-125 seeds and markers which are visible in Ultrasound/ X-ray/Fluoro/CT/MRI

- Best localization needles are available in 5, 7, 10 and 15 cm lengths
- Available in various ranges of seed activities with a shelf life of 90 days
- Needles are supplied sterile and are individually packaged and shielded



Loose Seed Loaded into Localization Needle



Seed in Tissue-Absorbable Strand Material

24 hours

We ship your order within 24 hours, 7 days a week Anywhere in the world!

Best[®] Fiducial Markers



Stranded

Loose

Best Medical can provide any standard or customized marker configuration made to your exact specifications, sterile or non-sterile!

Best Medical International, Inc. 7643 Fullerton Road, Springfield, VA 22153 USA
tel: 703 451 2378 800 336 4970 www.besttotalsolutions.com www.teambest.com

AFRICA | ASIA | EUROPE | LATIN AMERICA | MIDDLE EAST | NORTH AMERICA